CALEX® Cap Device

B-CALEX-C50

B-CALEX-C200

Revision: 2014-07-25

INSTRUCTION FOR USE

ENGLISH

INTENDED USE

With the BÜHLMANN CALEX® Cap device the user should be able to simultaneously collect a sample of stool material, without the need to manipulate this material with hands, and to homogenize, and dilute it in a way that the extract can be directly and quantitatively measured with BÜHLMANN fecal test systems.

INTENDED USER

Professional Use only.

Reagents	Content	Code	Preparation	Storage/Shelf life
CALEX® Cap Device	50	B-CALEX-C50	Ready to use	2-8 °C until the expiration date printed on the labels
filled with 5 mL extraction buffer	200	B-CALEX-C200		

SAFETY PRECAUTIONS

- Read the instruction carefully before you use the CALEX® Cap device.
- The extraction buffer does not contain components of human origin.
- The devices must be equilibrated at room temperature (18-28°C) before starting the extraction.
- The devices must not be used after the expiry date printed on the labels.
- The devices can be shipped at ambient temperature.
- The devices must not be reused for stool sample extraction.

HOW TO PROCEED WITH DIFFICULT STOOL SAMPLES

- **Liquid stool samples** can be pipetted directly into the CALEX® Cap device. Unscrew the blue cap and pipet 10 µl of stool sample into the device. Recap the CALEX® Cap device and proceed with step 4 on page 4.
- **Stool that does not stick to the grooves** can be sampled as follows: Position the stool sample on the wall of your stool collection tube or place it on a spatula/spoon. The dosing pin (refer to Fig. B, page 3) has to be pressed along the wall of the tube or against the surface of the spatula/spoon by circular movements until the grooves are completely filled with stool. Continue with step 3 on page 4.

INSTRUCTION FOR USE

Components of the device

Fig. A:

CALEX® Cap at delivery

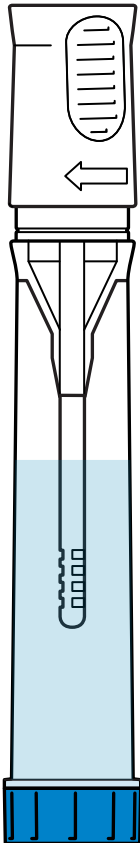
Fig. B:

Sampling pin


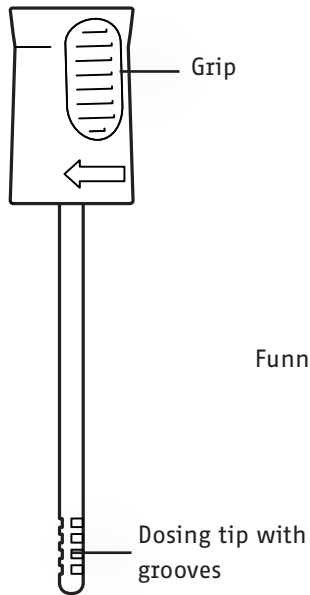
Fig C:

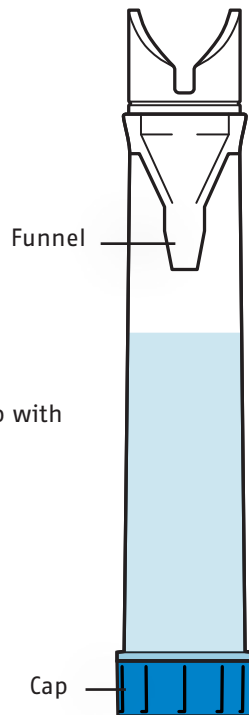CALEX® Cap body

INSTRUCTION FOR USE

1. Hold the CALEX® Cap device upright and remove the **white** sampling pin by simultaneously turning and pulling it upwards (picture **A**).

Do not unscrew the blue cap!

2. Dip the sampling pin with the grooves into the stool, turn the pin and remove it afterwards. Repeat the procedure at 3-5 different positions of the sample in order to fill the grooves completely (picture **B**).

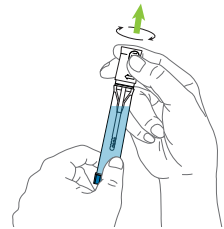
Excess stool is stripped off in the funnel during re-introduction of the sampling pin into the CALEX® body.

3. Re-introduce the sampling pin into the CALEX® body and push it back into the final locking position (you feel and hear two clicks, picture **C**).
4. Vortex the CALEX® Cap device with the white head down vigorously on a vortex mixer for 30 sec (picture **D**) and allow it to stand for 10 min on the white head. If the stool does not completely come off the grooves, repeat the procedure until the grooves are completely free of stool. That is easily monitored by checking that the grooves are empty (refer to fig. B, page 3) before you proceed with step 5.


Process the extract according to the instruction for use of the respective Calprotectin assay.

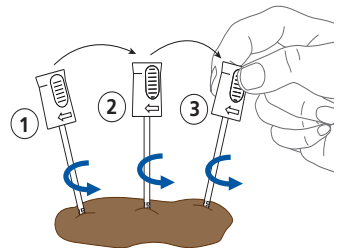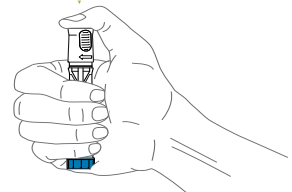
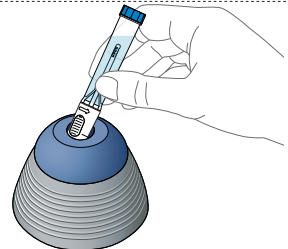
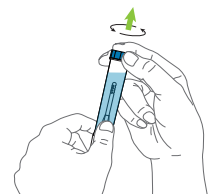
5. Unscrew the blue cap (picture **E**) and take the appropriate sample volume for the analysis; continue with the respective Calprotectin assay.
6. Tightly recap the CALEX® Cap device after usage.

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CALEX® is a registered trademark of BÜHLMANN in many countries of the world
BÜHLMANN Laboratories AG