


BÜHLMANN GanglioCombi™ MAG ELISA

Pipetting Template

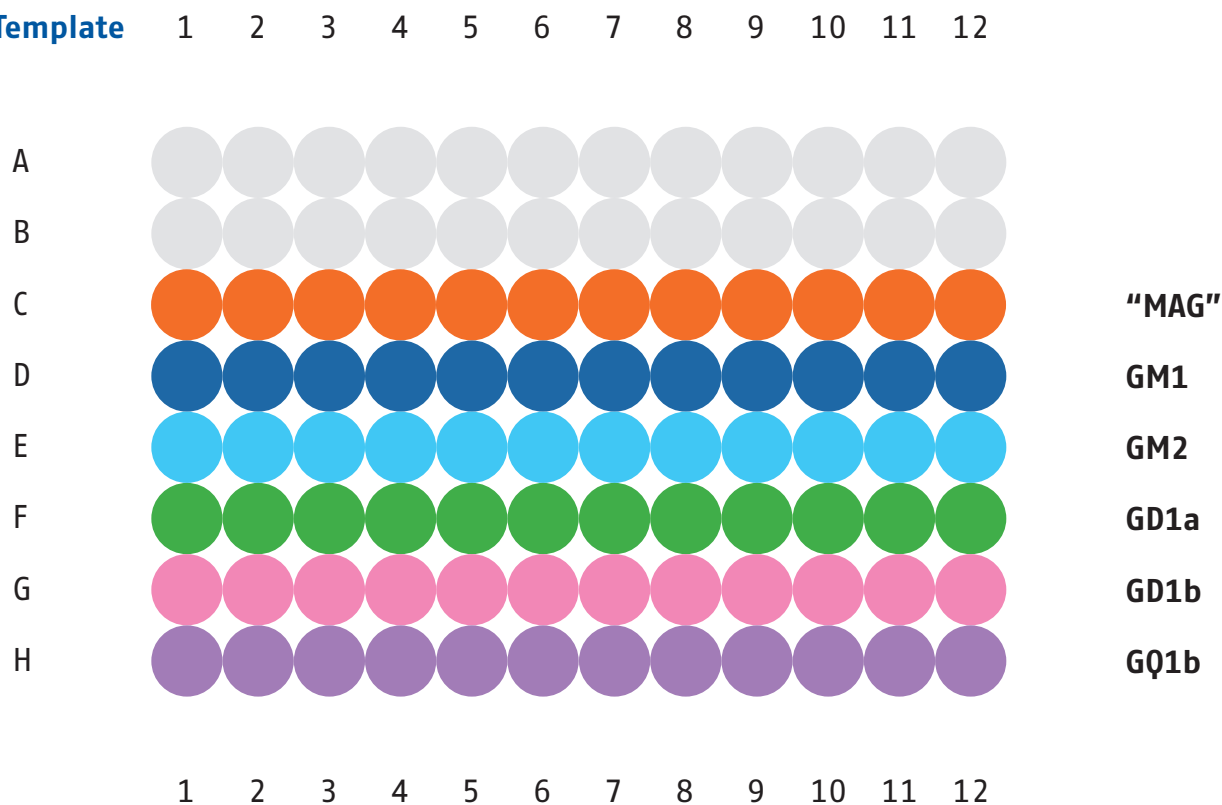


Pipetting Guidance


Precoated Microtiter Plate




100 µL of Calibrator, Controls or
Serum Samples (1:50)

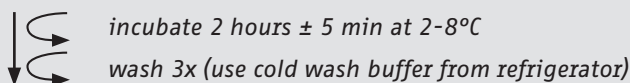

add 100 µL of Enzyme Label(s)

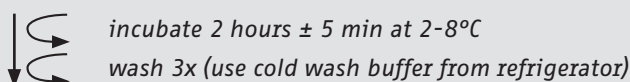


add 100 µL of TMB Substrate
(ambient temperature)

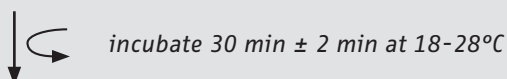

add 100 µL of Stop Solution

Read absorbance at 450 nm (within 30 minutes)

Autoimmune Peripheral Neuropathy: four easy Steps to complete the diagnostic assessment in your lab

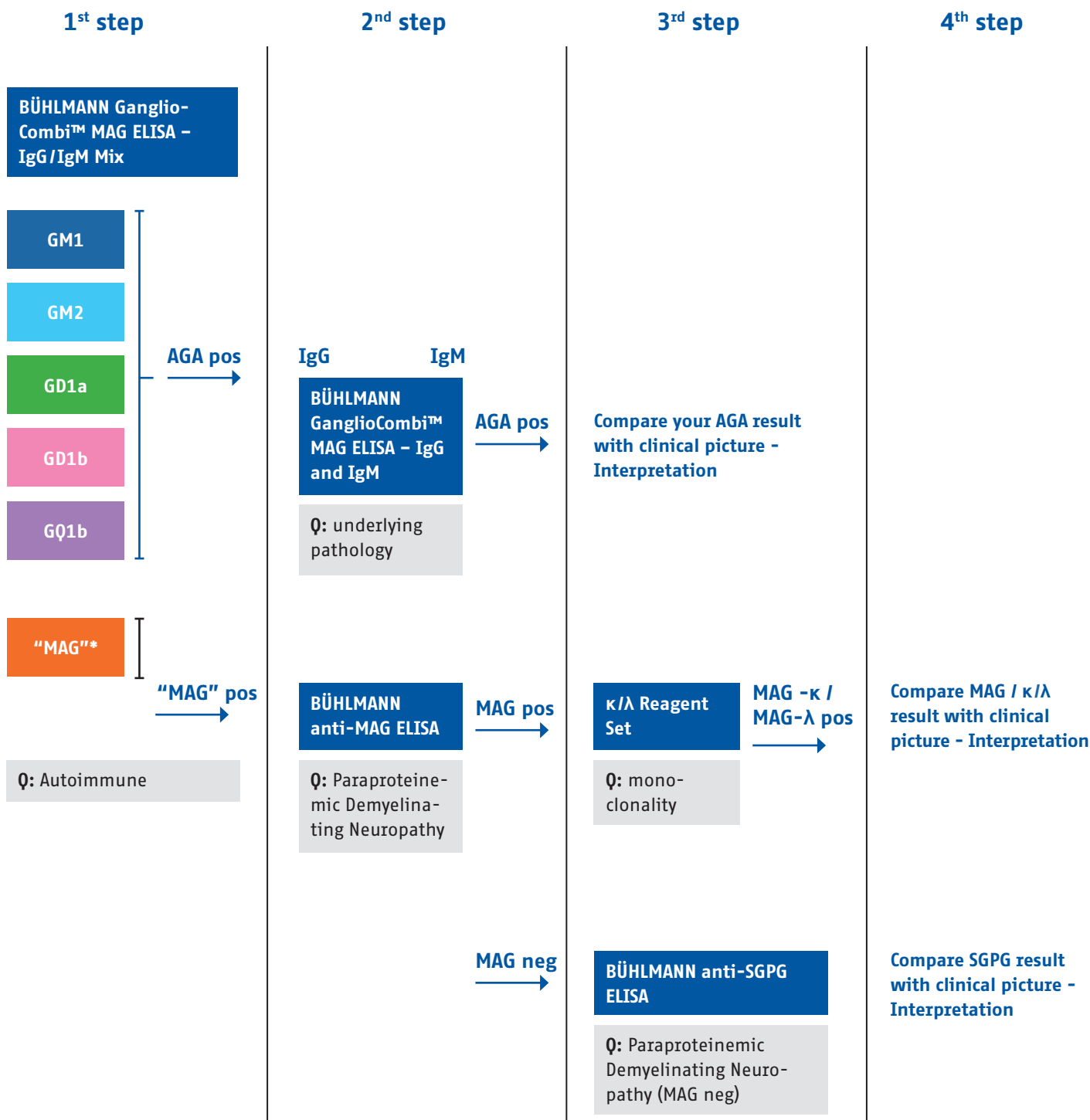

*„MAG“ is a chemically synthesized, sulfated disaccharide which mimics a MAG (HNK-1) epitope.

Abbreviation:
Q: Question



BÜHLMANN Laboratories AG
Baselstrasse 55
4124 Schönenbuch
Switzerland

Phone +41 61 487 12 12
Fax +41 61 487 12 34
info@buhlmannlabs.ch
www.buhlmannlabs.ch

BÜHLMANN GanglioCombi™ MAG ELISA
anti-MAG Autoantibodies ELISA
κ/λ Reagent Set
anti-SGPG Autoantibodies ELISA
BÜHLMANN GanglioCombi™ Light ELISA
anti-GM1 Autoantibodies ELISA

EK-GCM
EK-MAG
B-KLSET
EK-SGPG
EK-GCL-S
EK-GM1-GM